

SOFT SURGERY

Plexr is the Gold Standard in Soft Surgery treatments - non-invasive surgery: a treatment method performed through GMV medical devices alternative to classic surgery. Soft Surgery has been designed for all those who want to correct skin blemishes without resorting to surgery and / or pharmacological treatments: the results obtained are absolutely comparable to traditional methods for numerous treatments.

MINIMUM DISEASE FOR THE PATIENT

- Absence of risks and permanent side effects
- Very little perceived pain: 30 minutes before the treatment a local anesthetic cream is applied
- Rapid recovery time
- Lower costs compared to traditional surgery

MAIN APPLICATIONS

- Non-surgical blepharoplasty
- Wrinkle treatment
- Removal of fibroids, nevi, warts, keloids, xanthelasma, keratoses
- Active acne treatment
- Gummy smile treatment and gingival bleaching

Plexr[®] Plus
Made in Italy



CE0051

GMV



www.gruppogmv.com



Always choose maximum effectiveness and safety! This medical practice uses Plexr[®] by GMV, a medical device certified according to Directive 93/42/EEC for medical devices. Present for over 10 years on the market and distributed today in over 50 countries, Plexr[®] by GMV has introduced Plasma technology in Aesthetic Medicine and its applications are constantly studied and updated by doctors all over the world.

Plexr[®] Plus
Made in Italy

by
GMV

P
L
E
X
R
P
L
U
S

THE BREAKTHROUGH
OF PLASMA
IN AESTHETIC
MEDICINE





What is it for?

Plexr can treat a variety of skin imperfections, in particular by performing treatments for the removal of benign neoformations and elastic retraction treatments, thus generating a skin lifting.

How does it work?

Plexr is a device that ionizes the air present between its terminal and the epidermis, thus generating a micro-plasma beam. Plasma from the physical point of view is the fourth state of aggregation of matter, where the ions generated from molecules of neutral gas meet in the free state. In contact with the epidermis the micro-plasma beam sublimates * the corneocytes, ie the upper cells that form the protective layer of the epidermis. At the same time, an amount of energy selectively reaches the right depth of the dermis, without irradiation (laser / light) or electric shock (electrosurgical / current). This stimulation allows an immediate contraction and reorganization of the collagen fibers, a formation of new collagen and a renewal of the epidermal tissue.

* physical process whereby the evaporation of a substance, initially found in the solid state, occurs without passing from the liquid

How long is the treatment?

The duration of treatment is variable depending on the type of operation to be performed, but never exceeds 30 minutes, the time necessary for a non-surgical blepharoplasty. Other treatments, such as the removal of seborrhic keratosis, require only a few minutes.

Are there any contraindications?

There are no permanent side effects for treatments performed with Plexr by GMV by experienced and trained doctors. Scabs, edema and redness are part of the normal healing process and skin replacement. Normally this process lasts 7-20 days. It is however recommended to strictly follow the post treatment indications provided by the doctor.

Treating blemishes skin never was so simple: discover Plexr

How long do I see the results? Are they lasting?

The results are immediately visible and absolutely lasting over time. For some types of treatment it may take more than one session to maximize the effectiveness of the result.

Is it painful?

No. The treatment is practically painless, but if the patient is particularly sensitive, a small amount of anesthetic cream may be applied to the part to be treated 30 minutes before surgery.

Some treatments

Seborrhic wart



Before

After


Xanthelasma



Before

After

Wrinkles

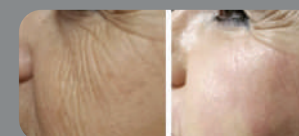

Before

After

Blepharoplasty



Before

After

